



XCELENS
extra + extreme HV
no pain fillers

exclusive formula


XCELENS + 0,3% LIDOCAINA

The experience and reliability already internationally recognized of research and development **swiss team Xcelens** is proud to introduce an innovative **Filler** that combines an excellent performance and results absolutely "No pain".

- ● ● ● ● **Xcelens Extra 3** and **Xcelens Extra 4** are formulated with a concentration of
- ● ● ● ● **23mg/ml** of hyaluronic acid cross-linked with lowest certified **BDDE** level on
- ● ● ● ● the market and with a percentage of **lidocaine at 0.3%**.
- ● ● ● ● **Xcelens Extra 3** and **Xcelens Extra 4** combine the safety of highly purified
- ● ● ● ● hyaluronic acid and cross-linked with **BDDE** to the high comfort of **lidocaine**
- ● ● ● ● contained therein.

+ MESH TECHNOLOGY

The most of the fillers present on the market usually get a cross-linking so-called "skein structure", instead the **Xcelens** range is made of highly cross-linked hyaluronic acid with a **cross-linking** so called "**mesh**", **Xcelens Extra 3** and **Xcelens Extra 4** have respectively 3 and 4 hyaluronic acid chains of different lengths in addition to "**free**" hyaluronic acid present into the innovative **Gel Xcelens**.

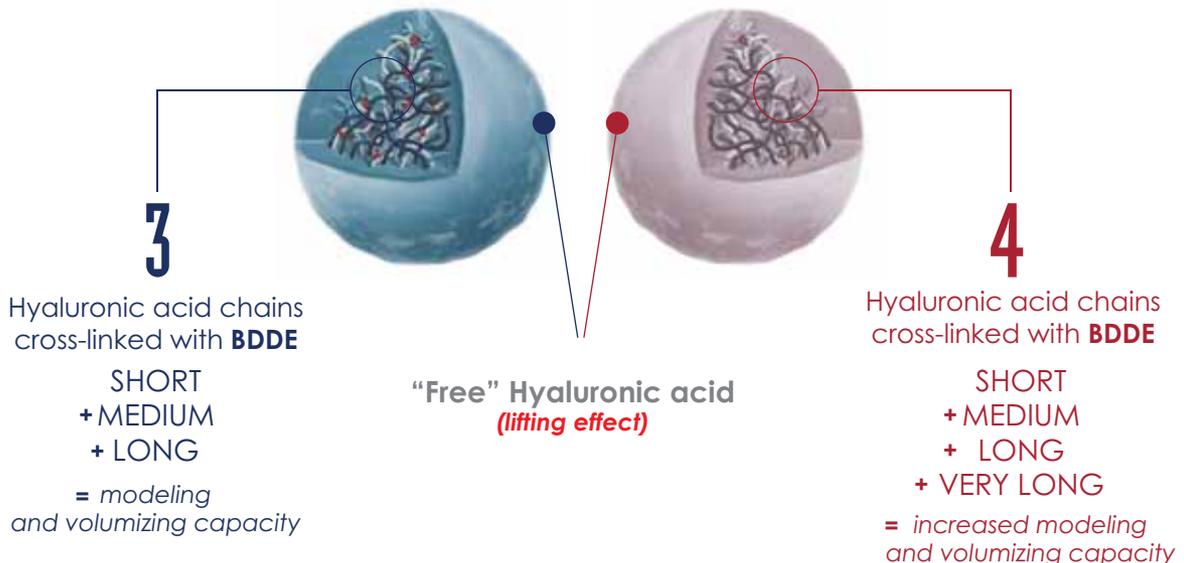



3

● ● ● ● ●	CONTENT	1ML
● ● ● ● ●	CONCENTRATION HA	23MG/ML
● ● ● ● ●	RETICULATING AGENT	BDDE
● ● ● ● ●	DEGREE OF CROSSLINKING	★★★★
● ● ● ● ●	NEEDLES	2 X 27G
● ● ● ● ●	LIDOCAINE	0,3%




TREATMENTS: fine lines, superficial wrinkles, glabellar lines, perioral wrinkles, crow's feet, labial commissure, moderate corrections to the nasolabial folds and lip contour.



4

● ● ● ● ●	CONTENT	1ML
● ● ● ● ●	CONCENTRATION HA	23MG/ML
● ● ● ● ●	RETICULATING AGENT	BDDE
● ● ● ● ●	DEGREE OF CROSSLINKING	★★★★★
● ● ● ● ●	NEEDLES	2 X 27G
● ● ● ● ●	LIDOCAINE	0,3%

TREATMENTS: Deep wrinkles and lines, jaw contour, chin and cheeks area, nasolabial folds, treatment of depressions, increase of lip volume and contour.

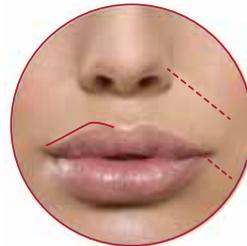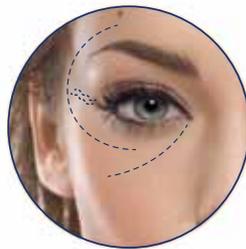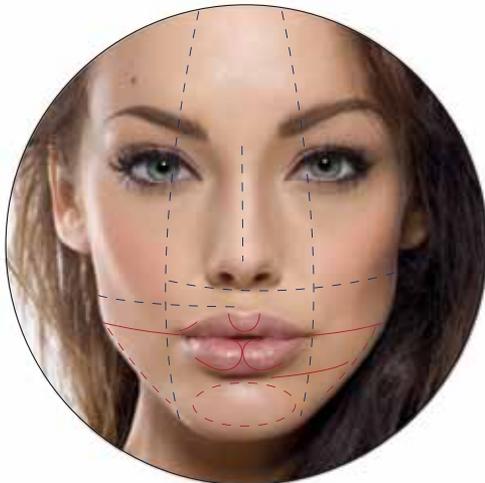


EXTRA 3 is indicated for the treatment of fine lines, especially for perioral wrinkles, crow's feet, glabellar lines, to penetrate the surface and middle stratum of dermis redefining their Appearance.

EXTRA 4 developed to fill deep and middle wrinkles, chin and cheeks area, jaw contour and small depressed areas. Viscosity, softness, elasticity are the main characteristics to keep a NATURAL appearance after the injection.

Extra 3 and **Extra 4** are often involved in the same areas. **Extra 3** increases the volume of lips and defines the shape of lip contour instead **Extra 4** has a volumizing effect for lips acting on the deep line and wrinkles.


immediate RESULTS + incredible DURATION

According to independent research centers in Italy and Switzerland, on a sample of 98 people, aged from 29 months to 72 years, the results obtained on the duration of the treatment allows us to ensure the product's effect has a significantly longer duration than other fillers on the market.

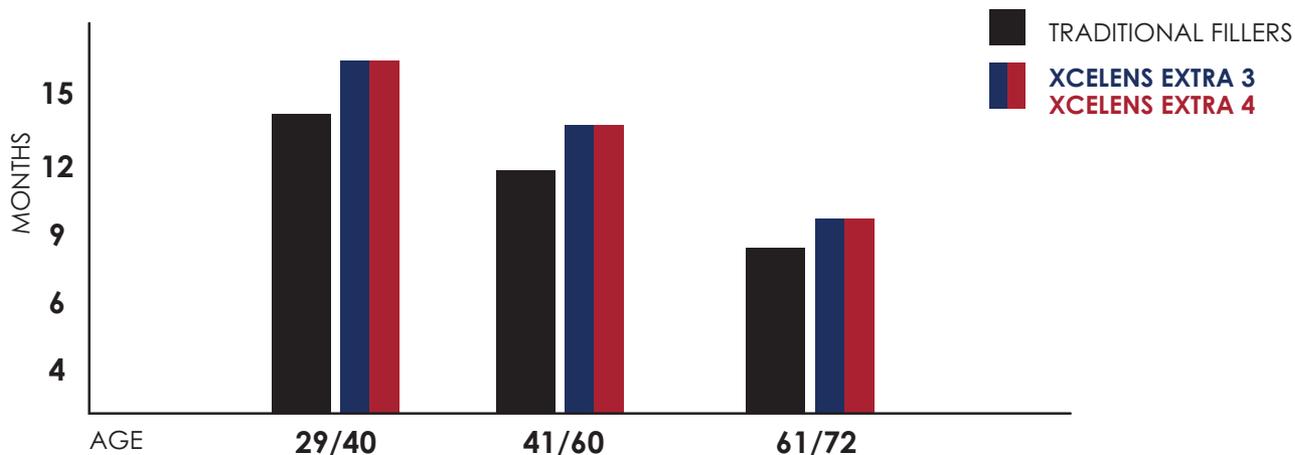

LESS PAIN + COMFORT


Thanks to the presence of LIDOCAINE 0.3%, the treatment will be safe, efficacious and painless, ensuring maximum comfort for the patient.

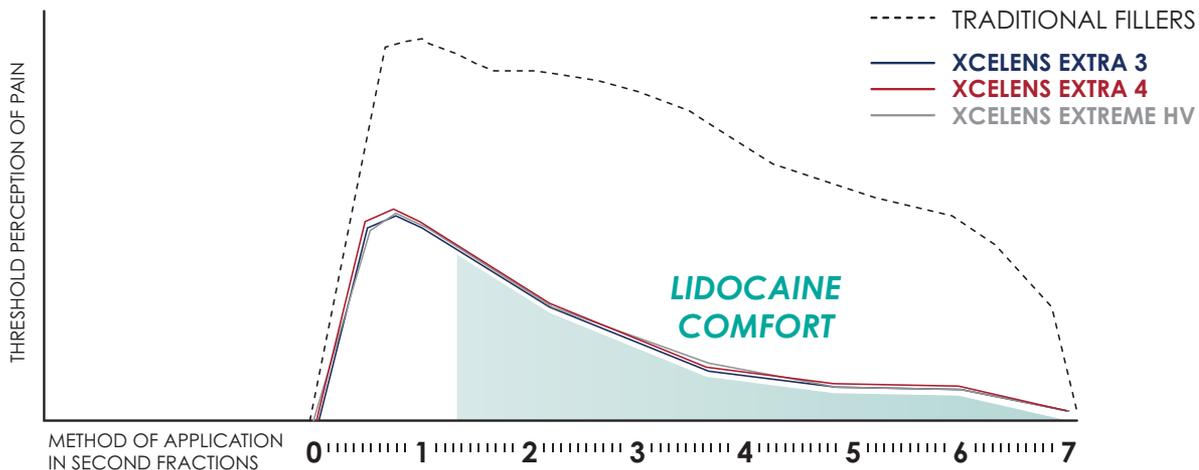

XCELENS EXTREME HIGH VISCOSITY


XCELENS EXTREME HV® the latest and innovative product with Xcelens exclusive formula to treat the lower third of the face.

XCELENS EXTREME HV® main target is to restore and reshape face contours, including the zygomatic area and more specifically the chin and jawline projection, sculpting and shaping the pogonion furrow (**angle G - Sn - Pog**).

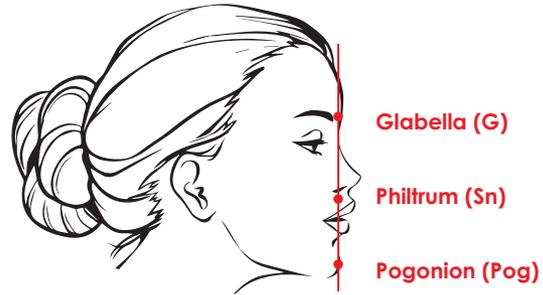

All comparative pre-clinical tests on face plus clinical studies of **XCELENS EXTREME HV®**, highlighted a duration of about 18/24 months; in addition, after 3 months from the implant., 100% of the doctors and 97.7% of the interviewed patients evaluated the face volumetric increase as "perfect" or "much better"

EFFECTIVENESS FOR MORE THAN 24 MONTHS

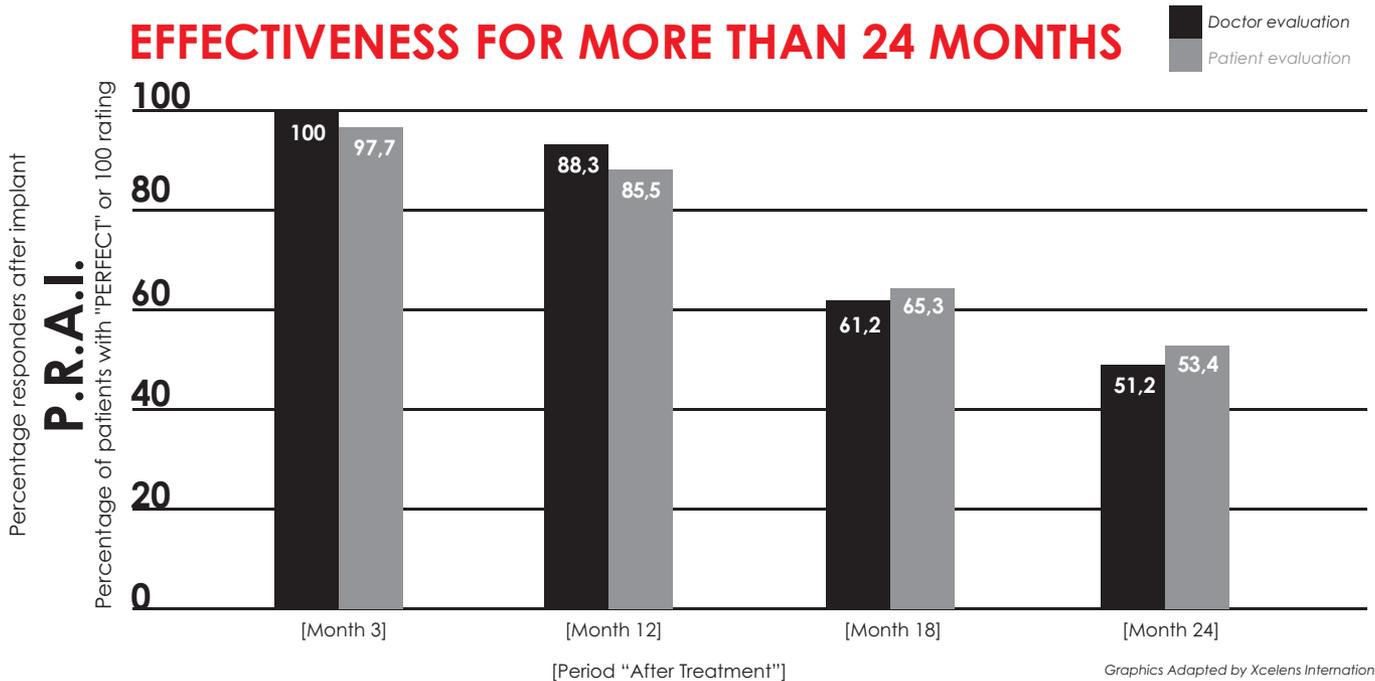

Prospective, randomized and controlled study of 100 patients treated exclusively on the 3rd lower of face from 1 to maximum 3 syringes per treatment (executed in maximum 2 visits - initial and retouch) with an interval of about 90 days. Patients returned for follow-up visits at months 1, 3, 6, 12, 18, 24.



EXTREME HV

XCELENS EXTREME HV® the ultimate available choice to reshape face contours restoring any volumetric deficit, including facial LIPOATROPHY. The exceptional viscoelastic properties of **XCELENS EXTREME HV®** combined with the new ergonomic and glass syringe give to doctors an incredible volumizing product together with an excellent hydrogel extrusion from the syringe.




**Patient satisfaction,
comfort and durability**

● ● ● ● ●	CONTENT	1ML
● ● ● ● ●	CONCENTRATION HA	25MG/ML
● ● ● ● ●	RETICULATING AGENT	BDDE
● ● ● ● ●	DEGREE OF CROSSLINKING	★★★★★
● ● ● ● ●	NEEDLES	2 X 27G
● ● ● ● ●	LIDOCAINE	0,3%

INDICATIONS: maxillary redefinition, zygomatic depressions, chin.

G PRIME [G']: rheology as the absolute judge

Rheology is the science that studies the flow and deformability of matter. G' is a rheological parameter indicating the filler capacity to return to its ordinary shape when exposed to dynamic forces. High value of G'Prime characterizes a hard, robust and compact gel with a better response to compression (greater elasticity). For this reason, a product with a greater G' is most suitable for thicker skin and deeper injections. Also an excellent extrusion of the hydrogel from the syringe.

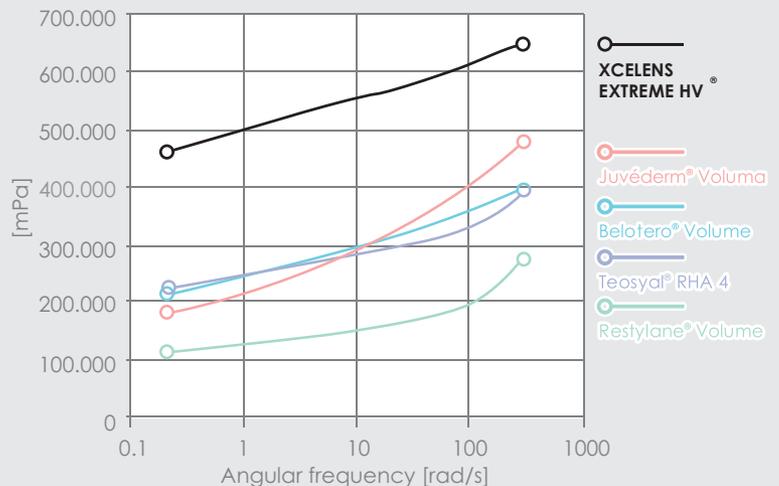

Values and physical-chemical characteristics, highlighted in the study of main fillers available worldwide, are the result of an independent research performed by Xcelens International.





follow us

 **Xcelens**  **xcelensgenyalofficial**

XCELENS INTERNATIONAL Sagl

Via Soldini 22 • Chiasso • CH 6830 • Switzerland
ph. +41 786.58.57.10 • info@xcelens.ch • **xcelens.ch**

